	 EN 374-5 : 2016	 VIRUS
GB	Micro-Organisms	Virus
FR	Micro-Organismes	Virus
DE	Mikroorganismen	Virus
ES	Microorganismos	Virus
IT	Microrganismo	Virus
PT	Micro-Organismos	Virus
NO	Mikroorganismer	Virus
DK	Mikroorganismer	Virus
SE	Mikroorganismer	Virus
NL	Micro-Organismes	Virus
FI	Mikro-Organismit	Virukset
GR	Μικροοργανισμοί	Ιός
TR	Mi k ro Organi z mal	Virüs
HU	Mikroorganizmusok	Vírus
EE	Mikroorganismid	Viirus
LV	Mikroorganismi	Viruss
HR	Djelomična Kemijska Zaštita	Virus
LT	Apsauga Nuo Mikroorganizmų	Virusai
BG	Μικροοργανισμοί	Вируси
PL	Mikroorganizmy	Wirusy
RO	Microorganisme	Viruși
SI	Mikroorganizmi	Virusi
SK	Mikroorganizmy	Vírusy
CZ	Mikroorganizmy	Virus
UA	Μικροοργανισμοί	Ускладнення
JP	微生物	ウイルス

	 U V W X Y Z	 X Y Z	
GB	Chemical risks		
FR	Risques chimiques		
DE	Chemische Gefahren		
ES	Riesgos químicos		
IT	Rischi chimici		
PT	Riscos químicos		
NO	Kjemiske risikoer		
DK	Kemiske risici		
SE	Kemiska risker		
NL	Chemische risico's		
FI	Kemialliset riskit		
GR	Χημική επικινδυνότητα		
TR	Kimyasal riskler		
HU	Vegyí veszélyforrások		
EE	Keemilised ohud		
LV	Ķīmiskie riski		
HR	Kemijski rizici		
LT	Cheminis pavojus		
BG	Химични опасности		
PL	Zagrożenia chemiczne		
RO	Riscuri chimice		
SI	Kemična tveganja		
SK	Chemické riziká		
CZ	Chemická rizika		
UA	Хімічні ризики		
JP	化学物質のリスク		

Disposable range

Instructions for use

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EN ISO 374-1
Type C

1 PRODUCT

** Performance level in accordance with EN ISO 374-1: 2016 +A1:2018 § 5.4.1		
Measured break through time (min)	Permeation performance level	Level acceptable for type
> 10	1	C
> 30	2	A/B/C
> 60	3	A/B/C
> 120	4	A/B/C
> 240	5	A/B/C
> 480	6	A/B/C

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C.T.C
4 rue Hermann Frenkel
69367 LYON Cedex 07 - France

ASQUAL
14 rue des Reculettes
75013 - Paris - France

SATRA - Technology centre Ltd
Wyndham Way, Telford Way, Kettering,
Northamptonshire, NN16 8SD - United Kingdom

GB / DISPOSABLE RANGE FIELD OF APPLICATION

- These gloves comply with the requirements for protection against splashes, minor chemical hazards, and microorganisms, against radioactive contamination for glove 980. Resistance to penetration was evaluated under laboratory conditions and only concerns the test specimen under investigation.
- These gloves do not protect against mechanical risks.
- The CE marking on these products means that they meet the requirements of EU Regulation 2016/425 on Personal Protective Equipment concerning protection, comfort and strength. Gloves meet the requirements (innocuousness, comfort, robustness and protection against the risks claimed) of the PPE regulation 2016/425. The CE marking (module B) was issued by CTC notified body 0075. For the gloves which meet the requirements of PPE regulation (EU) 2016/425 on personal protective equipment, as amended to apply in GB, the UKCA marking was issued by SATRA technology centre Ltd (AB0321).
- The permeation levels obtained do not reflect the actual protection time in the work place, nor the difference between mixtures and pure chemicals.
- Chemical resistance was assessed under laboratory conditions from samples taken only from the palm and concerns only the chemical being tested. It may be different if it is used in a mixture.
- Gloves giving protection from radioactive contamination do not protect from ionising radiation and have not undergone the stress crack resistance test under the effect of ozone. They are not designed to be used in containment enclosures. They may be used as an under-glove for handling waste and for routine cleaning work.
- The gloves do not contain substances at levels such as are known or suspected to have harmful effects on the hygiene or health of the user under foreseeable conditions of use.
- For category III gloves - Protection against fatal or irreversible hazards: Module D, monitored by ASQUAL- 0334.
- For category III gloves - Protection against fatal or irreversible hazards: Module D, monitored by SATRA UK - 0321. Module C2, monitored by CTC NB 0075.

INSTRUCTIONS FOR STORAGE AND USE

- Ambidextrous gloves for single use only, should be stored in their original packaging and kept away from heat.
- The design performances are not significantly affected by ageing if the gloves are stored in the appropriate conditions (humidity, temperature, cleanliness, ventilation, lighting).
- For gloves containing natural latex: avoid contact with oils and petroleum, aromatic or chlorinated solvents.
- For Gloves containing nitrile and/or neoprene and/or synthetic material: avoid contact with ketones and organic nitrogen products.
- For PVC gloves: avoid contact with ketones and aromatic or chlorinated solvents.
- It is recommended that you pre-test the gloves as the actual workplace conditions of use may differ from those of the CE type tests (in particular mechanical and/or chemical), according to temperature, abrasion and degradation. Tests carried out on the palm.
- When they are used, the protective gloves offer less resistance to hazardous chemical products because of the alteration of their physical properties. Movements, snags, rubbing or degradation caused by contact with chemical products, etc. can considerably reduce the gloves' real service life.
- For corrosive chemical products, degradation can be the most important factor to take into account when choosing gloves resistant to chemical products. Before use, it is recommended that the gloves be inspected so as to ensure that they do not have any fault or imperfection.
- Make sure that hands are clean and dry before putting the gloves on.
- Not recommended for use by people who are sensitive to dithiocarbamates and thiazoles. For gloves containing natural latex: not recommended for use by those sensitive to natural latex and thiuram.
- To avoid contaminating the hand with a dirty glove, be sure to remove it by pinching it from the base of the wrist to the fingers. Slide your fingers into the second glove and remove the glove. Avoid touching the skin.
- For more information about performance levels, chemical resistance and usage of the gloves, please contact your distributor or MAPA PROFESSIONAL Technical Customer Support.
- Information leaflet, EU/UKCA Declaration of Conformity can be downloaded from www.mapa-pro.co.uk

Newell Brands UK Ltd, Halifax Ave, Fradley Park, Lichfield, Staffs, WS13 8SS, UK.
T : (44) 1 905 450300 – F : (44) 1 905 450350 – DG 1 905 450360. www.mapa-pro.co.uk

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